

granulGEL

granulated conductive solid gel
for EEG recording



- Long-term stable low impedance
- Quality EEG signal as close to EEG on liquid gel
- Clean and dry head hair
 - no need to wash hair after EEG examination
- Easy electrode and cap cleaning
 - need less time on electrodes washing after EEG examination
 - less aggressive wash of electrodes increase their durability
- Comfort for operator and for patient

ADVANTAGES

GranulGEL is compatible with electrodes MCScap-E,
electrode caps MCScap BASE and ELECTRODE SET.



MCScap BASE

Electrode cap with pre-installed electrodes and common connector

MCScap ELECTRODE SET

Electrode set of caps and electrodes with single connectors



Use instructions:

1. Put the cap with fixing rings on the head
2. Prepare skin by cotton stick with EEG scrub
3. Part hair under each electrode placement if needs
4. Insert granuGEL in the fixing rings and after put electrodes in



granuGEL
unpacking and installation



How preparation is affected on impedance:

